BEST VISION INTERNATIONAL Actimart
1140 Rue André Ampère, CS80544
13594 AIX-EN-PROVENCE Cedex3
FRANCE
TEL : 04 88 14 81 34
FAX : 04 88 14 81 36
E-MAIL : contact@bvi-varionet.com
WEBSITE: www.bvi-varionet.com

BVI MODEL: VH10

PRE-MOUNTED SPECTACLES AND EYE PROTECTOR TO CORRECT PRESBYOPIA

USER INSTRUCTIONS ACCORDING TO EN ISO 16034: 2002 / EN 14139: 2010 - REGULATION (UE) 2017/745

This safety eyewear is in conformity with the provisions of Regulation (EU) 2017/745 on class 1 medical devices and is in conformity with standard EN ISO 16034: 2002 / EN 14139: 2010. This safety eyewear is intended for near vision only.

It is not suitable for regular use without the approval of an eyecare professional. Do not use in case of difficulties in adapting to the powered lenses (headaches, blurred vision, eyestrain) and seek professional advice. Pupillary distance (PD) of 62 mm. It is not suitable for driving or vehicle operation. It is not suitable for far vision.

USER INSTRUCTIONS ACCORDING TO EN 166: 2001 - REGULATION (UE) 2016/425

EC – Type certification, issue by: 2849 - INSPEC International B.V. - Bechwege 54-62 1119 PW - Schiphol-Rijk - The Netherlands. This BVI safety eyewear model satisfies the requirements of the Regulation (EU) 2016/425, regarding equipment for individual protection and is made according to the requirements of the EN 166: 2001.

APPLICATIONS: This BVI safety standard eyewear is in accordance with the reference standards, the CE marking and the number of the EN standard is compulsory with the various symbols of the using domain. Best Vision International identification: **BVI**

Increase robustness	S	Increased robustness (12mvs)
High speed particles symbols:	F	Low energy impact (45mvs)
(lens and frame marking)	B	Medium energy impact (120mvs)
	A	High energy impact (190mvs)

The letter T immediately after the impact letter allows a use for high speed particles at extremes of temperature. In the absence of the letter T the safety eyewear must be in ambient temperature.

Symbol for the optical class: (lens marking)	2	Intermittent work
	3	Occasional – warning: not suitable for long term use
Applications symbols:	3	Liquids droplets or splashes (frame)
	4	Large dust particles >5 microns (frame)
	5	Classes and small dust particles <5 microns (frame)
	6	Short circuit electric arc (frame and lens)
	9	Molten metal and hot solids (frame and lens)
Standards by application (lens marking)		
	EN 169 "Welding filters"	Code - 1.7 to 16
	EN 170 "Ultraviolet filters"	2* 1.2 to 5
	EN 171 "Infrared filters"	4* 1.2 to 10
	EN 172 "Solar protection filters without infrared specification"	5** 1.1 to 4.1
	EN 172 "Solar protection filters with infrared specification"	6 1.1 to 4.1
	"Symbol C": good colour recognition	
	Resistance to surface damage by fine particles	K
	Resistance to fogging	N
	Original lens	O
	Spare lens	∇

**** NOTE:** Sun tinted is in conformance with Standard EN 172 Code n° 5, Shade n° 3.1 – allows more than 8% light transmission, but less than 17.8%.

WARNINGS

Eyeprotectors against high speed particles worn over standard optalmic spectacles can pass on the impact with a potential risk to the wearer. Inspect the eye protector regularly: pitted or scratched lenses or damaged frames may seriously reduce protection and will require replacement with a new eye protector. Replacement lenses are not available. If the symbols F,B and A are not common to both the ocular and the frame then it is the lower level which shall be assigned to the complete eye-protector. For a face-shield to comply with field of use symbol 8 it shall be fitted with a field of scale 2-1,2 or 3-1,2 and have a minimum thickness of 1,4mm. For an eye-protector to comply with field of use symbol 9 both the frame and ocular shall be marked with this symbol together with one of the symbols F,B or A. Materials which come into contact with the wearer's skin could cause allergic reaction to susceptible individuals.

PRECAUTIONS OF USE

Cleaning and disinfection: product must be cleaned or disinfected with a product like soapy water, optical cleaning solution, or a solution very low in alcohol. **Maintenance:** Like all other protective products, the life depends on the type of use, care and maintenance. If your frame is damaged, replace the entire article, except when a temple is damaged. A damaged temple can be replaced with a new one, provided by BVI upon request. We recommend that spectacles are kept in a suitable case when not in use. In every case, the product must be replaced after a period of 2 years of use. **Transport:** The protective eyewear must be transported in its packing of origin. **Storage:** store between 5 to 40 °C; humidity less than 90%.

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